

# BOOTS & CABLES PROGRAMMING CONNECTION

PRODUCT*	BTE 13 (Power and Power Plus)	BTE 13 (Standard)	BTE 312	RIC 13	RIC 312	micro RIC 312 Wireless	Custom
<b>Z SERIES™</b>	Orange Flex Strip 	Purple 13 Boot 	Black 312 Boot 	N/A	Black 312 Boot 	Orange Flex Strip 	Orange Flex Strip 
<b>HALO™</b>	N/A	Cable (straight) 	N/A	Orange Flex Strip 	N/A	N/A	N/A
<b>3 SERIES™</b>	Orange Flex Strip 	Purple 13 Boot 	Black 312 Boot 	N/A	Black 312 Boot 	N/A	Orange Flex Strip 
<b>XINO™</b>	N/A	N/A	N/A	N/A	N/A	Orange Flex Strip 	N/A
<b>SOUNDLENS® 2</b>	N/A	N/A	N/A	N/A	N/A	N/A	Orange Flex Strip

## ADDITIONAL NOTES

- Inspire® will display any discovered wireless devices. Detected wireless devices must be manually selected prior to initiating the fitting session.
- VC position does not matter as the external volume control is disabled during programming.
- Starkey uses purple programming cables. Flex Strips and all boots will connect to the cables.
- Wireless Programming requires use of either the SurfLink or TruLink Programmer as indicated.

\* Pictures shown are the most recent product casing.

## GOVERNMENT SERVICES

AMERICAN OWNED ★ AMERICAN OPERATED



- Product can be programmed using SurfLink® Programmer



- Product can be programmed using TruLink™ Programmer

## IMPORTANT

### WHEN PROGRAMMING:

- Run Performance Update with a wired connection.
- Open and close the battery door of the hearing device after programming is complete.

# BOOTS & CABLES PROGRAMMING CONNECTION CHART

## HALO



RIC 13




BTE 13

## XINO



Wireless Micro RIC 312

## SOUNDLENS 2

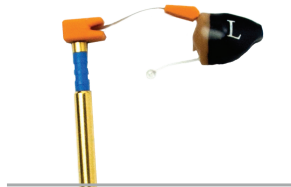

IIC

## Z SERIES AND 3 SERIES



RIC 312



Wireless Micro RIC 312




Mini BTE 312



BTE 13



Power Plus BTE 13

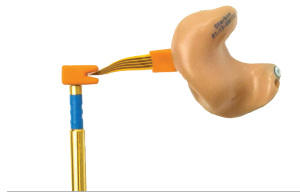

ITE



ITC



CIC