

SURFLINK REMOTE CONTROL


PRODUCT AVAILABILITY: Picasso, Muse iQ, SoundLens Synergy iQ, Muse, SoundLens Synergy, Z Series Wireless, 3 Series Wireless, and Xino Wireless Devices

Launch **Inspire® X**, then connect hearing devices. Refer to the *Initial Fit Protocol QuickTips* for additional information.

SETUP IN INSPIRE X

1. Select the **Accessories** tab from the left navigation bar.
2. Select the check box next to **Remote Control**.
3. Under **Sync**, enter the four character address located on the back of the remote into the field on the screen, then select **Sync**.
4. Select the **Favorite Button Configuration**, if using the Advanced remote control. Configuration options include:
 - » **None**
 - » **Stop/Start Audio Stream:** Default
 - » **Comfort Boost:** An instantaneous, aggressive noise management setting
 - » **Jump to Memory 4**
 - » **Tinnitus On/Off**

The SurfLink Remote Control allows users to make adjustments to their 900MHz hearing devices. It is available in Basic and Advanced configurations.

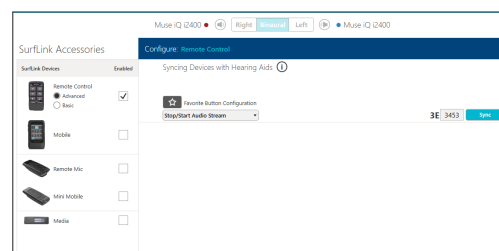

Setup in Inspire X

USE

- » **Unlock:** Unlock to make adjustments. Lock to prevent inadvertent changes and preserve battery life.
- » **Ear Selection Switch:** Keep in the center for binaural adjustments. Slide to the right or left for individual-sided adjustments.
- » **Volume:** The up arrow increases volume, the down arrow decreases volume.
- » **Memory:** Cycles through all memories.
- » **Home:** Returns hearing device to power-on volume and memory settings.
- » **Favorite Button:** Activates associated Favorite Button configuration (see above).



Basic (left) and Advanced (right)